Potrebno upoštevati navodila za uporabo/ Consult instructions for use/ Bitte beachten Sie die Bedienungsanleitung/ Potrebno poštivati uputstva za uporabu/ Viz návod k použití/ Необходимо учитывать инструкции по применению/ Қолдану бойынша нұсқаулықты ескеру қажет/ Consultar las instrucciones de uso/ Dodržujte návod na použitie

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EN

CC DISK PMMA Transparent is used in CAD/CAM milling machines for making PMMA structures for casting metal frameworks and for modelling press ceramic works. The milled structure can be tested in the patient's mouth. The product is 100 % organic and burns out without residues.

Indications:

- Milled modelling for metal structure casting.
- Milled modelling for pressed ceramics.
- Testing of milled modelling in the patient's mouth before casting.
- Testing of milled modelling before making zirconium framework.

Contraindications: Known allergies to PMMA.

Intended users: The product is intended to be used by professionals – dental technicians.

Patient selection criteria: Totally or partially edentulous patients.

Final safety notes: Since processing and polishing particles are produced, it is recommendable to use protective gloves and protective masks with FFP1 filters.

Side effects: In very rare cases, a patient may develop an allergy to PMMA. The dentist is responsible for selecting the suitable material.

Storage: Store at room temperature.

Notice: Any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Warranty: Whether given verbally, in writing, or by practical instructions, our recommendations for use are based upon our own experience and trials, and can only be considered as reference values. Our products are subject to further development. Therefore, alterations in the structure and composition are reserved.

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